

Vacuum mats for patient positioning during surgery



- ? Conforms to each patients body and body part
- ? Removes pressure points
- ? Quick, safe and easy positioning

Great benefits:

- ? Effective prevention of decubitus and peresis
- ? Optimum access for the surgeon to the operating site
- ? Excellent patient comfort
- ? Improved efficiency in the operating room
- ? Making Inter-clinic transportation in defined surgical position possible
- ? Can be used as reusable alternative to plaster beds

resulting in GREAT COST SAVINGS



Indications to use the MedVac vacuum positioning system during surgery:

- ? Side position
- ? Half-side position
- ? Back position
- ? Sitting / half- sitting position
- ? Coiled position
- ? Lithotomy position
- ? Laparoscopic positions

How to operate the MedVac system:

- ? Make sure the inner material is evenly distributed and the vacuum mat is flat
- ? Vacuum the mat slightly
- ? Place the patient on the mat
- ? Open the valve to let air into the mat making it soft
- ? Put the patient in the required position
- ? Vacuum the mat until firm

Features of all MedVac products:

Special high quality polyurethane film:

- ? Durable, colourfast material guarantees long life
Soft as silk on the patient still extremely tough and puncture resistant
- ? Clean design without places that can trap fluids and dirt makes it very easy to clean and disinfect.
- ? Biocompatible, PVC and latex free


Automatic vacuum valve:

- ? Proven, robust low profile vacuum valve that automatically closes when suction stops.
No risk of losing vacuum if slipping with the vacuum pump


Compatible with X-ray, CT and MRI. CE certified

Labels on every product:

- ? Individual label with ID-number
- ? Label with Instruction how to operate the valve
- ? Label with distributor logo and information (optional)
- ? Label with customer logo and information (optional)

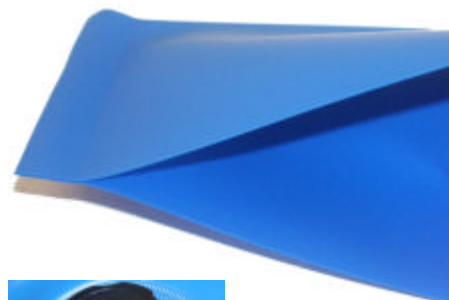

MedVac surgical vacuum mat range:

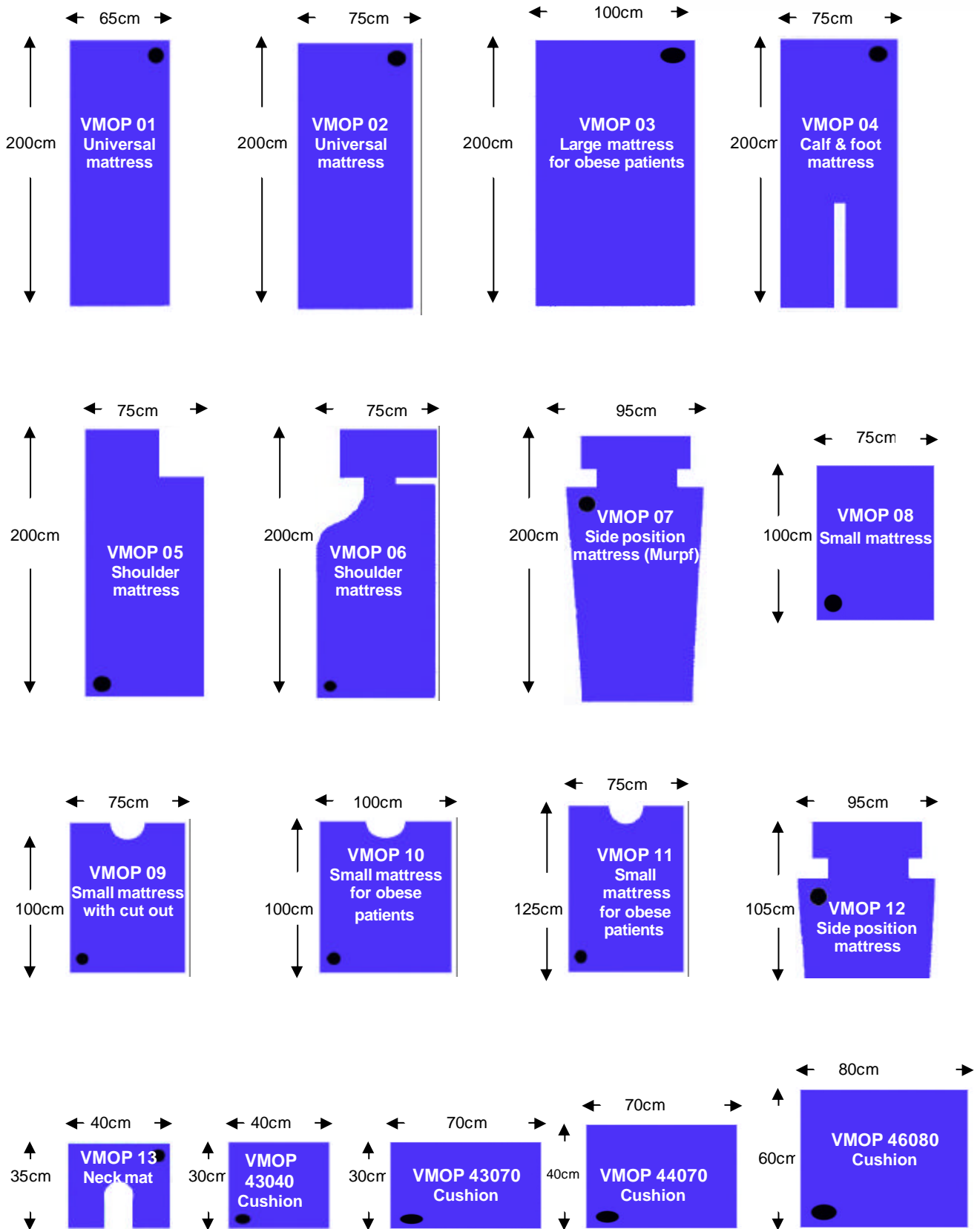

Kohlbrat & Bunz GmbH Loretostrasse 4-8 A-5550 Radstadt

Phone: +43 6452 7193 0 Fax: +43 6452 7193 51 Email: office@kohlbrat-bunz.com www.redvac.com



- made by Kohlbrat & Bunz